


Converting CASMED Monitors from Male Slip Luers to Bayonet Quick-Connect Fittings Option #1:



If your CAS monitor is more than 3 years old, you may have cuffs that do not fit the hose. Easy to use adapters are available to upgrade the hose for the new cuffs.



Insert the adapter # ZFT-RX1ML.

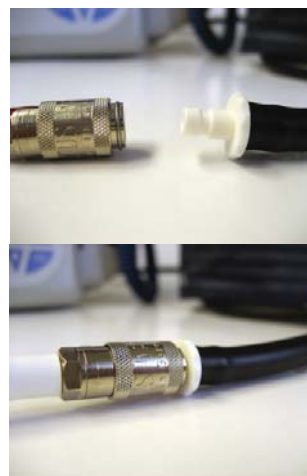

Now you can use your older monitors with new cuffs!



Converting CASMED Monitors from Male Slip Luers to Bayonet Quick-Connect Fittings

Option #2:



If your CAS monitor is more than 3 years old, you may have cuffs that do not fit the hose. Instead of using adapters, your facility may opt to cut the fitting from the end of the monitor hose.



Cut off the original fitting and insert the new fitting # 27-03-0104.



Now you can use your older
monitors with new cuffs!

